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GB:  
Mounting of accessories for the multi-function scales

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
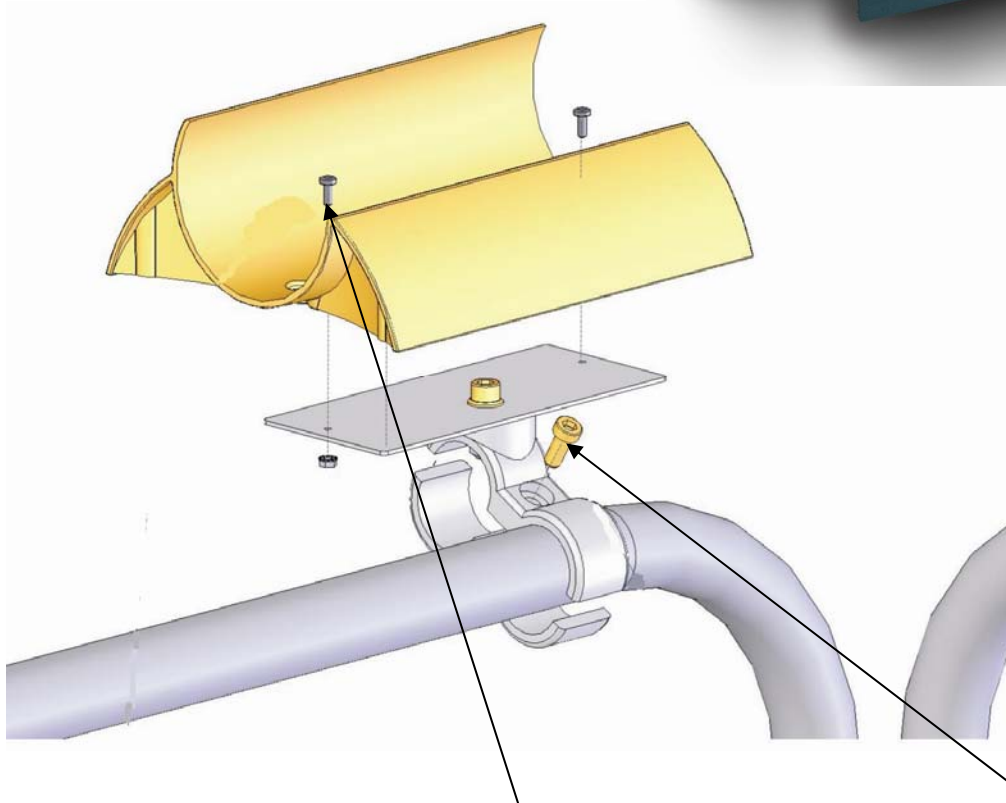
# Multi-function Scale - Mounting

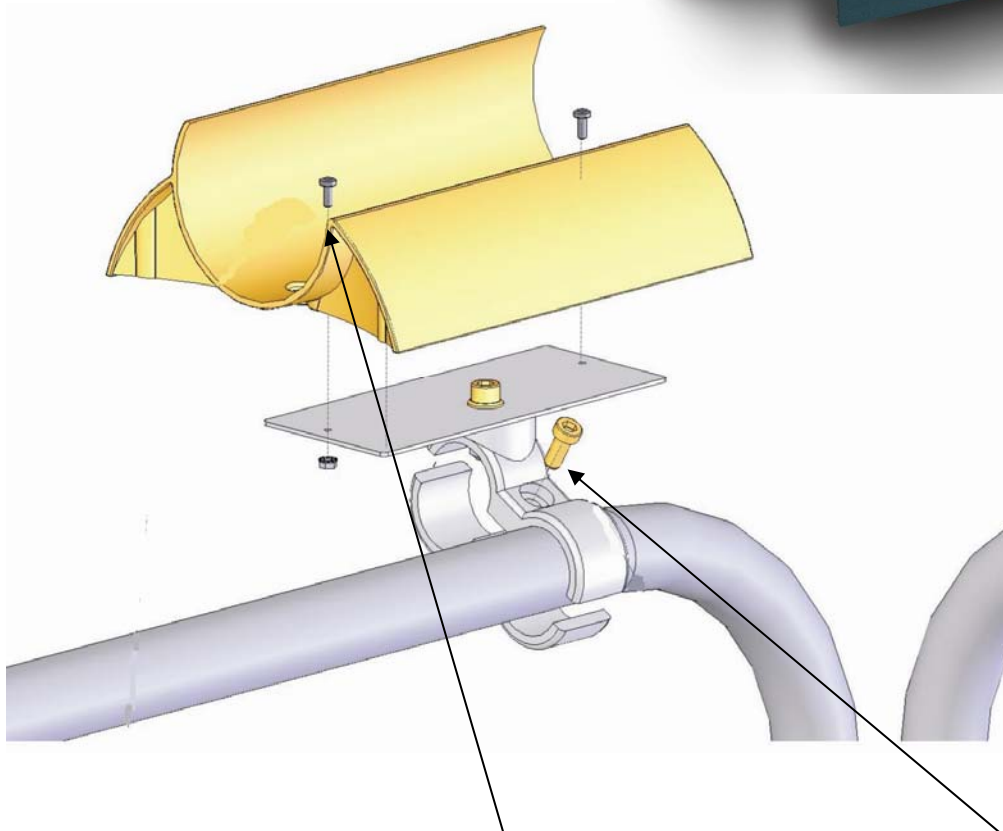German | english | French

## Instruction for mounting the indicator 2790

Screw the holder of the indicator with the upper part of the railing connection as follows.

Pass M6 screw with washer through the wall bracket and the intermediate washer, bolt screw with the upper bonding, set screw connection that a slight rotation is still existent to prevent loosening the screw, locking the cap nut on the bottom of the screw tightly.

Tighten the connection on the railing with wrench; mount the aligning on horizontal position.

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
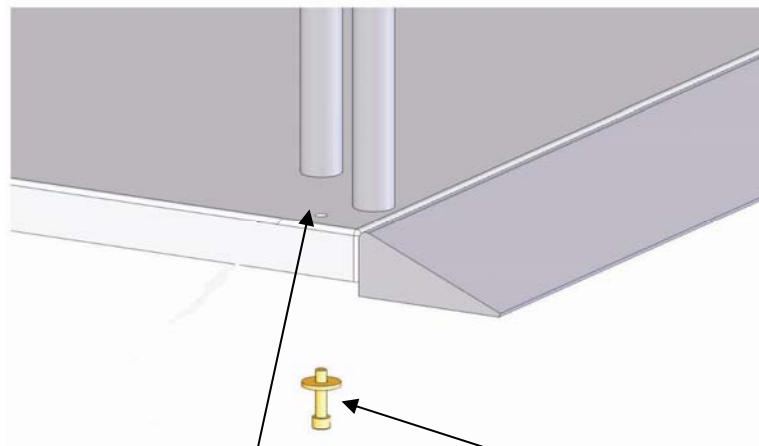
## Mounting instructions for the railing Ref.: 2530.03.001, 2530.04.001

Remove plastic plugs.

Position the railing on the upper side of the platform.

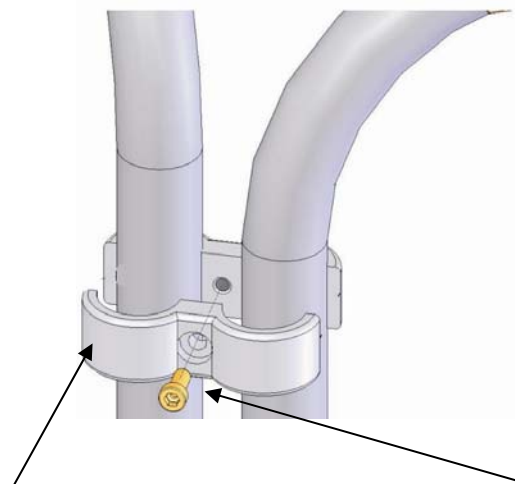Pass M8 screw with washer from the inside through the fixing hole; tighten screws with the included pin wrench.

Mount the railing connection in the upper section of the railing, by this connection you get a stable construction.

Tighten the railing with the pin wrench.

The pipe connector is not in the scope of delivery 2530.03.001, this must be done separately with the No. 2510.07.001 be ordered.

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

# Multifunction Scale - Mounting

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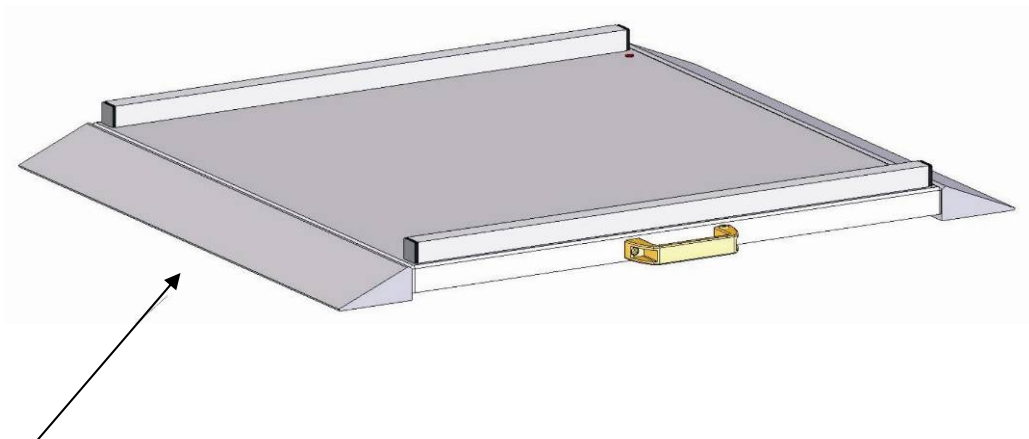## Mounting instructions for the drive-on ramp Ref.: 2505.08.001

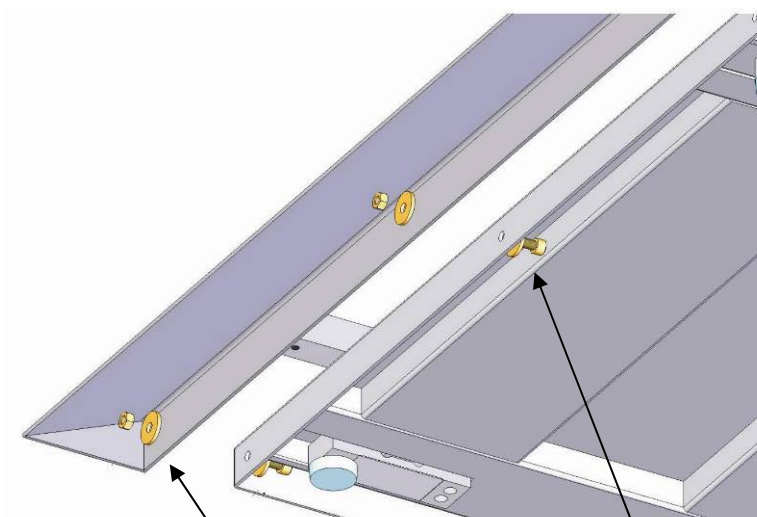Preferred site for assembling the first ramp; to the left of the handle.

Remove plastic  
plugs.


Position the ramp at  
the front.

Pass M8 screw with washer  
from the inside through  
the fixing hole of the  
drive-on ramp; place in  
washer and screw hex-nut  
M8, tighten screws with  
the included pin wrench.

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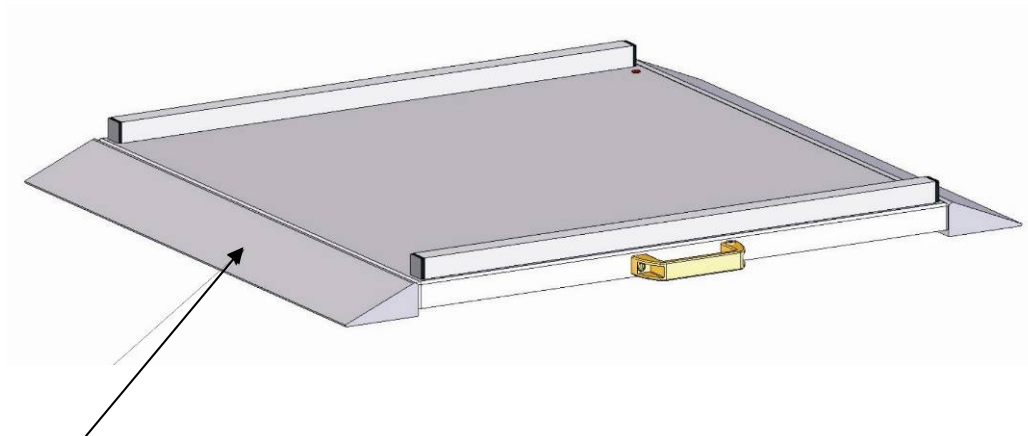
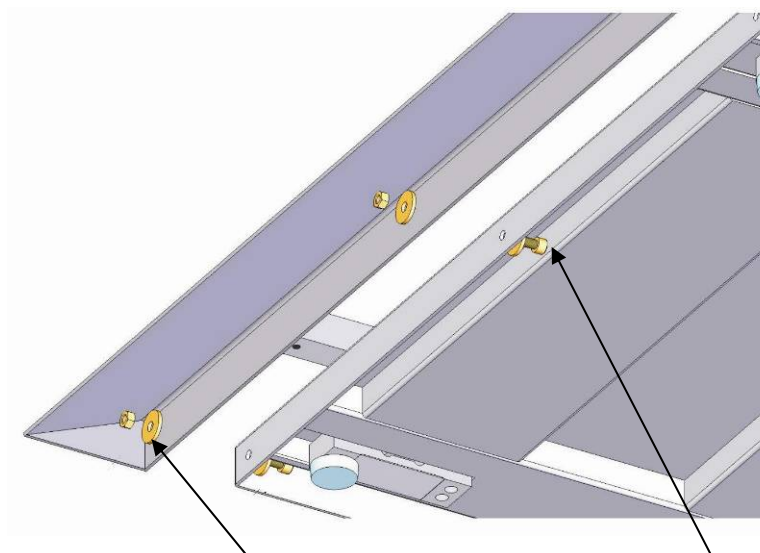
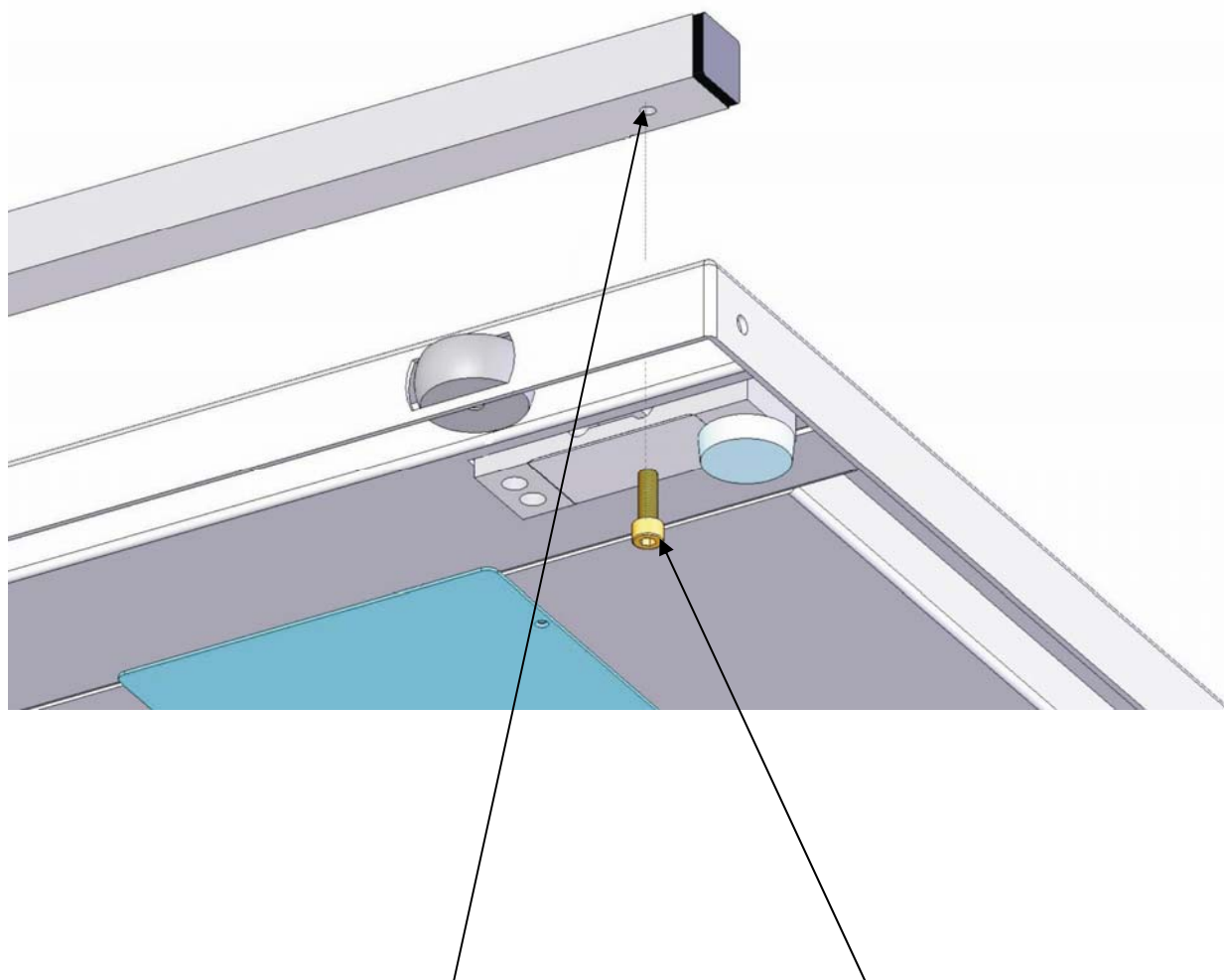
# Multifunction Scale - Mounting

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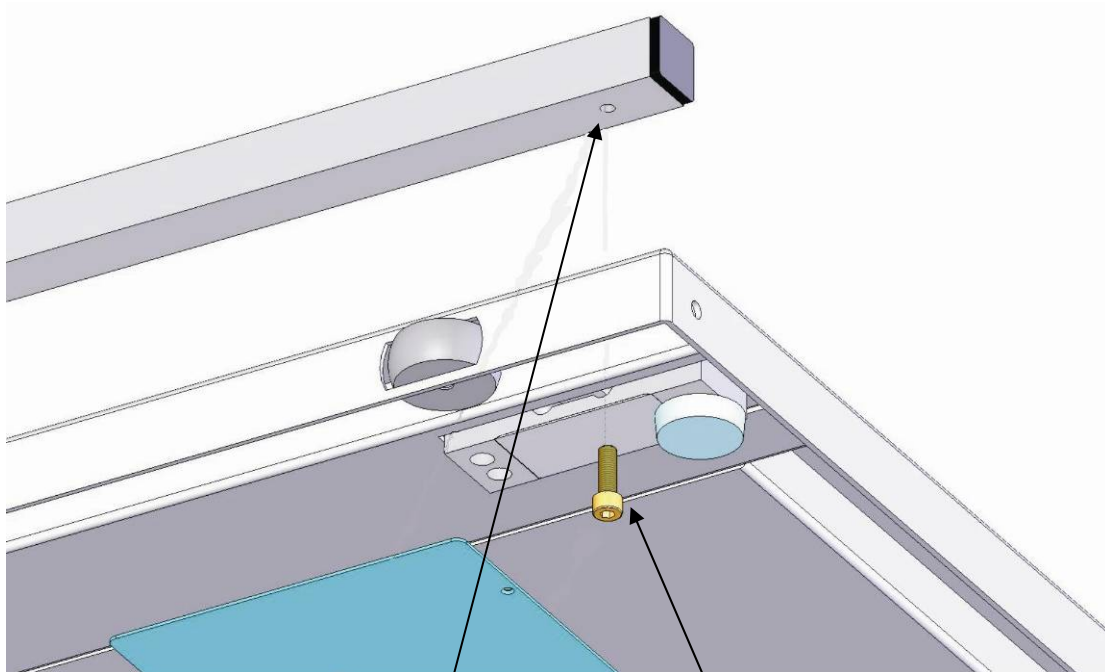## Mounting instruction for the side guide Ref.: 2502.04.001

Remove plastic  
plugs.

Position the side  
guide on the  
platform.

Pass M8 screw with  
washer from the inside  
through the fixing hole;  
tighten screws with the  
included pin wrench.

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