**Montage**

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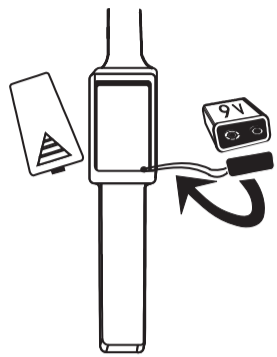
Mounting

Fit upper section with measuring sensor on display unit. Secure with screw provided.

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Battery

Use an alkaline battery (9 V block, 6 LR 61). This will increase the number of possible measurements.

Setting-up

Place 9 V block battery in the battery compartment located at the rear of the control unit.

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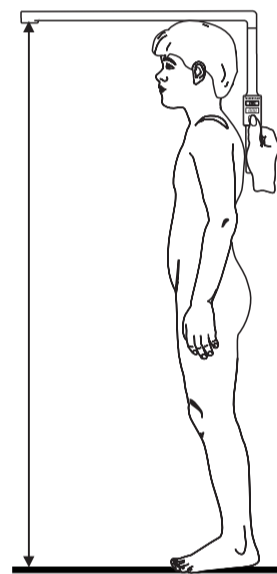
Selecting unit of measure

The height measuring unit can display measured values in "cm" or "inch".
With the unit switched off, press and hold the "ON" button for longer than 10 seconds until the display flashes.
Change between "cm" and "inch" display by briefly pressing the button.
Then press and hold the "ON" button for approx. 10 seconds and your selected unit of measure will now be set.

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Measuring

The person to be measured should stand upright with head held straight on a clear, smooth and even surface.
Place the height measuring unit on the head of the person and hold it exactly in horizontal position with the aid of the spirit level.
Make sure there is no interruption between the sensor and the floor.
Now press the "ON" button. The test digits "888" first appear in the display. Hold the measuring unit completely still, a signal sounds to confirm measurement is completed.
The measured value is displayed for approx. 30 seconds.
To make subsequent measurements, reset the unit to the new measuring position and press the "ON" button.

Attention

Carpet may falsify the measuring result.

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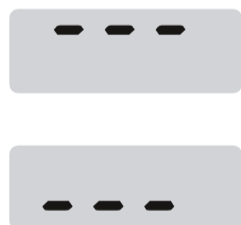
Switching off

The unit is switched off automatically after approx. 30 seconds.

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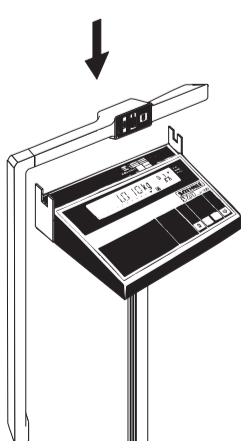
Replace battery

Dispose of used batteries as special waste.

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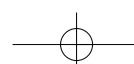
Assembling of the brackets for S20-indicators

Attach the supplied brackets on the left and on the right side of the indicator. Put on the height measuring unit.

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This device is radio interference suppressed in compliance with valid EC Regulations 89/336/EEC.

Note: Under extreme electromagnetic influences, e.g. operation of a radio set in the immediate vicinity of the measuring unit, the displayed value may be adversely effected. Once the source of interference has been removed, the unit can be used for its intended purpose, it may be necessary to switch on the measuring unit again.

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**Mounting and Operating instructions
Height Measuring Unit**

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